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2019/05

LOT 10083 A REF 395W.01

This solution is useable for all biotecnica analyzer's series (BT1000,BT1500,BT2000,BT3000,BT3500,BT4500)

Produced by Far Asa Med Company Co.ltd

Extraordinary washing must be done by the operator weekly or if there is a message on monitor.the procedures are described below: Through the analyzer menu or the specific icon ,access to the function 'Analyzer Utilities' and then select the function "Extera Wash cuvettes".Then follow the instructions on the screen.Start the washing procedure with R1 solution.At the end of the first washing procedure with solution R1 run second washing procedure with solution R2.

Tel : + 982183374
www.Farasamed.com

biotecnica instruments

Extera wash solution

Acid Reagent **R1**

250 mL
(50mL free)

2019/05

LOT 10085 A REF 395W.02

This solution is useable for all biotecnica analyzer's series (BT1000,BT1500,BT2000,BT3000,BT3500,BT4500)

Produced by Far Asa Med Company Co.ltd

Extraordinary washing must be done by the operator weekly or if there is a message on monitor.the procedures are described below: Through the analyzer menu or the specific icon ,access to the function 'Analyzer Utilities' and then select the function "Extera Wash cuvettes".Then follow the instructions on the screen.Start the washing procedure with R1 solution.At the end of the first washing procedure with solution R1 run second washing procedure with solution R2.

Tel : + 982183374
www.Farasamed.com

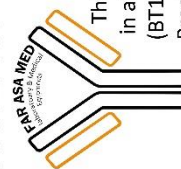
biotecnica instruments

TENSIOATTIVO CONC
CONC SURFACE AGENT

100 ml

Lot: 08273 A

2019-09

This solution is useable for deionized water

in all Biotecnica analyzer's series
(BT1000,BT1500,BT2000,BT3000,BT3500,BT4500)

Produced by Far Asa Med Company Co.ltd

CAUTION:

Do not touch directly.

Only use for Biochemistry Analyzer.

It is necessary to use this solution 1ml for 1litre water in the instruments

The solution is prepared and it can be used directly in the water.

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